# TRIXIE

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**(EN) First Aid**  
for dogs and cats

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## EN Contents

Contents of First Aid Kit.....	7
Standard values and their definition.....	7
Shock .....	8
Open Wounds .....	9
How to put on a short-time paw bandage.....	9
How to put on a compression bandage.....	10

## Contents of First Aid Kit

	Contents	Possible application
1	bag with handle	easy to store in car or backpack
2	elastic gauze bandage (3x)	to securely attach wound dressings to frequently moving or rounded body parts
3	self-adhesive bandage, narrow (1x)	to attach and compress wound dressings
4	self-adhesive bandage, wide (1x)	to attach and compress wound dressings
5	roll of plaster, low adhesive strength, (1x)	to attach the end of a wound dressing on the fur, can be removed more easily and with less pain
6	roll of plaster, high adhesive strength, (1x)	to attach the end of a wound dressing on the dressing itself
7	roll of padded wadding, narrow (1x)	for padding wounds
8	roll of padded wadding, wide (1x)	for padding wounds
9	bandage scissors (1x)	to individually adapt wound dressing materials
10	gauze pads, absorbent (10x)	to stop the bleeding, clean wounds as well as apply disinfectants
11	alcohol pads (10x) (70 % Isopropyl Alcohol)	for sterile wound cleaning

For pictures see p. 2

## Standard Values and their Definitions

In order to determine abnormalities, it is necessary to know the standards of healthy pets. Therefore it is advisable to train with healthy pets. It is very helpful to gain experience in an emergency situation. The pet gets used to the treatment which helps decrease the chance of biased standards caused by stress or anxiety.

**Standard values to be checked:**

- 1. Respiration:** The respiratory rate can be controlled at the thorax.
- 2. Pulse:** The pulse is taken on the inside of the thigh. Do not use a thumb to check for a pulse because it has its own pulse (see picture page 3).
- 3. Colour of the mucous membrane:** A healthy mucous membrane is pale pink. If the mucous membrane gets a blue tinge, this indicates oxygen deficiency (see picture page 3).

4. **Capillary filling time (CF):** In order to control the capillary filling time press on a pigment-free spot of the gums or the mucous membrane of the upper lip. After letting go, only 2 seconds later this spot should be back to its normal colour.

Standard values of cats and dogs	dog	cat
respiration (gasps/min)	15–30	20–40
pulse (rate/min)	70–150	80–140
temperature (to be measured rectally)	37.5–39.0 °C	38.0–39.3 °C
CF	2 sek	2 sek

A higher pulse rate is typical for smaller and younger pets. The temperature of puppies and kittens can be up to 0.5 °C respectively 0.2 °C above normal.

## Shock

A shock is an extreme circulatory disturbance which can become life-threatening. The blood does not circulate normally but first supplies vital organs like the brain, lungs and heart. This phase can only be maintained for a short period. If the state of shock lasts longer the pet could collapse.

### Possible reasons:

- heavy bleeding
- lasting diarrhoea or vomiting (leads to enormous fluid loss)
- extreme anxiety or shock

### Symptoms of collapse:

- increasing lethargy of the pet
- rising pulse rate but decreasing pulse intensity
- increasing respiration rate (rapid but shallow respiration)
- pale mucous membranes
- tongue becomes bluish from lack of oxygen
- capillary filling time is more than 2 seconds
- pet body temperature decreases (noticeable at the limbs)

In case of shock the pet should be taken to the vet as soon as possible, where the pet will be provided with hydration infusions and other specific medical treatment.

### **Steps for the pet owner in an emergency:**

1. **Calm** the pet down.
2. Keep your pet warm **by putting on a blanket.**
3. **How to lay your pet in case of shock:** Put the pet's back on a slightly raised pad so that the blood reaches heart, lungs and brain easily.

If your pet gets too stressed or is in too much pain, please do not practise the shock position.

4. Heavy bleeding should be stopped with a **compression bandage** (see picture page 6).

Moving the pet for transport has to be done with care since every change of position will put heavy strain on the pet's blood circulation.

## Open Wounds

Wounds which are tended to quickly heal well. How to treat them depends on the type of wound.

### Bites

Bites are contaminated by germs which are located in every pet's mouth and this can lead to blood poisoning in the most serious cases. Therefore it is advisable to see the vet, in order to prevent the wound from getting infected but to treat it appropriately.

### Abrasions (injury to the outer layer of skin)

Let the wound bleed slightly so that impurities can be washed out. In case of large impurities the wound should be disinfected. Normally abrasions heal by themselves within a few days. A little protective bandage or a veterinary collar can prevent contamination by germs or irritation from licking.

### Cuts

Cuts are often located in the paw area. This happens e.g. when a pet steps on a piece of broken glass. Due to permanent contact with the soil impurities cannot be avoided. Therefore the paw should be covered and bandaged sterilely. Before bandaging, impurities such as shards of glass should be removed carefully with tweezers.

## How to put on a short-time paw bandage

Bandages on paws must always be padded sufficiently (at least between toes and pads). Do not apply the bandage too tightly as that could interrupt the blood circulation. In case of swelling above or below the bandage, loosen or remove the bandage immediately. Please consult a vet if the swelling does not go down.

Begin below the paw and pull the bandage to the front over the claws and then over the whole paw. Next wind the bandage roll around the leg in circular movements, but not too tightly (see pictures page 6). Finally take the self-adhesive bandage and repeat the procedure. For deep wounds the injured pet should be taken to the vet.

## How to put on a compression bandage

Heavy bleeding should be stopped by a compression bandage. The wound has to be covered by a sterile wound compress and be bandaged several times around. In

order to get consistent pressure on the bleeding area a special compression pad e.g. a pack of bandages has to be put on it (see picture page 6). Please make sure that the bandage is not too tight, so that blood supply to the extremities is not cut off. If the extremity swells and gets cold the bandage has to be loosened.

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